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## Foreword

---

The **FLUXMEDICARE® Protect** covers are accessories exclusively designed for the protection of the light-emitting textiles against leaks or dirt that could occur during the treatment with the FLUXMEDICARE® system (please refer to the User Manual of the **FLUXMEDICARE®** medical device).

These nonwoven laminated covers keep the light-emitting textile clean and offer a better hygiene during the use.

The **FLUXMEDICARE® Protect** covers are especially designed for a single use with the different models of the **FLUXMEDICARE®** light emitting textiles. They have to be thrown away after each use.




The **FLUXMEDICARE® Protect** covers are a CE marked Class I medical devices in accordance with Regulation (EU) 2017/745. Therapeutic indications, contraindication, target of patients, users, location and environment of use, precautions and warnings are the same that the ones for the **FLUXMEDICARE®** system (refer to the user manual of the **FLUXMEDICARE®** medical device).

## 1. Composition

The **FLUXMEDICARE® Protect** cover are made of 28 gr/sqm polypropylen PP nonwoven with a one-side laminated polyethylen PE film. Oeko-Tex® 100 certified.

## 2. Sizes of the protective cover

The **FLUXMEDICARE® Protect** covers are available in several sizes to fit the different light emitting textiles models of the **FLUXMEDICARE®** medical device.

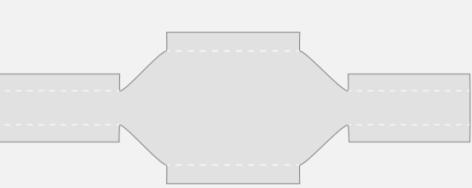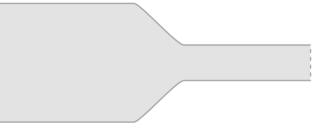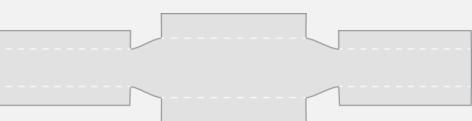
Reference	Description	TEXTLIGHT picture	Use	FLUXMEDICARE® Protect picture*
FMP01	Non-woven protective cover for light-emitting textile TEXTLIGHT1 – Single size		Single use	
FMP02	Non-woven protective cover for light-emitting textile TEXTLIGHT2 – Single size		Single use	
FMP03	Non-woven protective cover for light-emitting textile TEXTLIGHT3 – Single size		Single use	
FMP04	Non-woven protective cover for light-emitting textile TEXTLIGHT4 – Single size		Single use	
FMP06	Non-woven protective cover for light-emitting textile TEXTLIGHT6 – Single size		Single use	

\* Non contractual illustrations


### 3. Set up of the protective covers on the FLUXMEDICARE® light-emitting textile

#### 3.1. Protective « FLUXMEDICARE® Protect » cover 1 for light-emitting textile T1


The protective « FLUXMEDICARE® Protect » cover - ref FMP01 – is especially designed for the **FLUXMEDICARE® T1 light-emitting textile (Textlight1)**.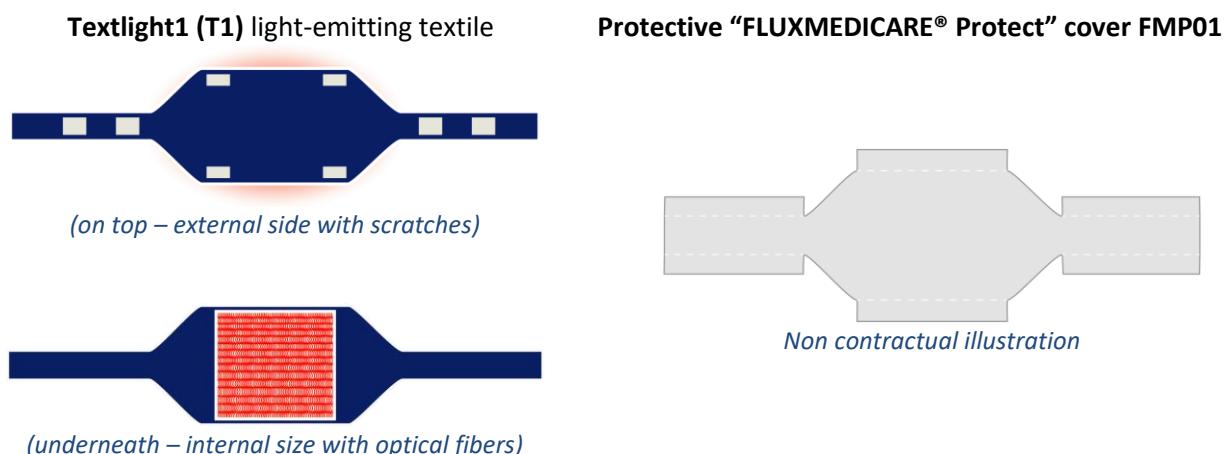

##### STEP 1 : central part of the protective cover

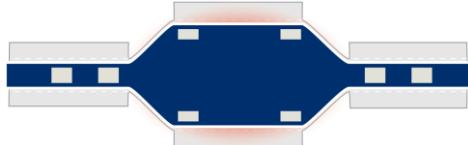- 1) Lay flat the protective cover on a plane surface, smooth plastic side outward.
- 2) Put the light emitting textile (Textlight 1 model) on the cover. The navy fabric side with scratches must be put on the outside. The white light emitting band with optical fibers must be put underneath, not visible, and must be in close contact with the cover.

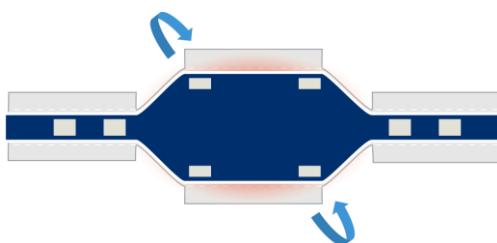- 3) Close the flaps of the central part of the cover on the textile. Secure with the scratches. Adjust if necessary.

##### STEP 2 : lateral parts

Close the flaps of the lateral parts of the cover on each end of the textile. Secure with the scratches. Adjust.

##### END OF SET-UP

The Textlight1 light emitting textile with the FLUXMEDICARE® Protect cover ref FMP01 is now ready for use.

*(Please refer to the user manual of the FLUXMEDICARE® medical device).*

The protective cover is for single use; it must be thrown away after each use.


### **3.2. Protective « FLUXMEDICARE® Protect » cover 2 for light-emitting textile T2**

The protective « **FLUXMEDICARE® Protect** » cover - ref **FMP02** – is especially designed for the **FLUXMEDICARE® T2 light-emitting textile (Textlight2)**.

**Textlight2 (T2) light-emitting textile**

**Protective “FLUXMEDICARE® Protect” cover FMP02**

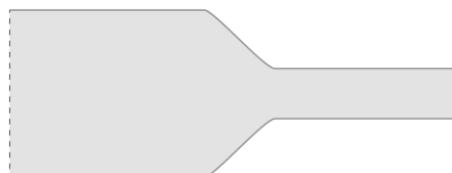*Non contractual illustration*

**STEP 1 : Put on the protective cover**

**END OF SET-UP**

The Textlight2 light emitting textile with **FLUXMEDICARE® Protect** cover ref **FMP02** is now ready for use.

*(Please refer to the user manual of the **FLUXMEDICARE®** medical device).*

The protective cover is for single use; it must be thrown away after each use.

### 3.3. Protective « FLUXMEDICARE® Protect » cover 3 for light-emitting textile T3

The protective « FLUXMEDICARE® Protect » cover - ref FMP03 – is especially designed for the **FLUXMEDICARE® T3 light-emitting textile (Textlight3)**.

**Textlight3 (T3) light-emitting textile**

**Protective “FLUXMEDICARE® Protect” cover FMP03**

*Non contractual illustration*

**STEP 1 : Put on the protective cover**

**END OF SET-UP**

The Textlight3 light emitting textile with FLUXMEDICARE® Protect cover ref FMP03 is now ready for use.

(Please refer to the user manual of the FLUXMEDICARE® medical device).

The protective cover is for single use; it must be thrown away after each use.

### 3.4. Protective « FLUXMEDICARE® Protect » covers 4 & 6 for light-emitting textiles T4 & T6

The protective « FLUXMEDICARE® Protect » cover - ref FMP04 – is especially designed for the FLUXMEDICARE® T4 light-emitting textile (Textlight4) and the protective « FLUXMEDICARE® Protect » cover - ref FMP06 – is especially designed for the FLUXMEDICARE® T6 light-emitting textile (Textlight6).

**Textlight4 (T4) light-emitting textile**

(on top – external side with scratches)

**Textlight6 (T6) light-emitting textile**

(on top – external side with scratches)

(underneath – internal side with optical fibers)

**Protective “FLUXMEDICARE® Protect” cover**

**FMP04 or FMP06**


*Non contractual illustration*

**STEP 1 : central part of the protective cover**


- 1) Lay flat the protective cover on a plane surface, smooth plastic side outward
- 2) Put the light emitting textile (Textlight4 model) on the cover. The navy fabric side with scratches must be put on the outside. The white light emitting band with optical fibers must be put underneath, not visible, and must be in close contact with the cover.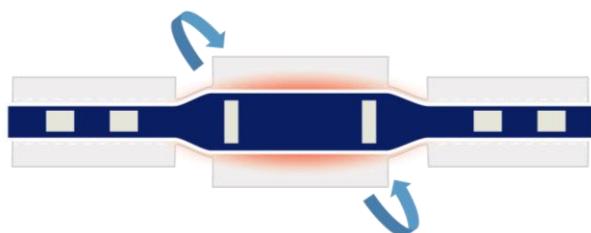


- 3) Close the flaps of the central part of the cover on the textile. Secure with the scratches. Adjust if necessary.

**STEP 2 : lateral parts**

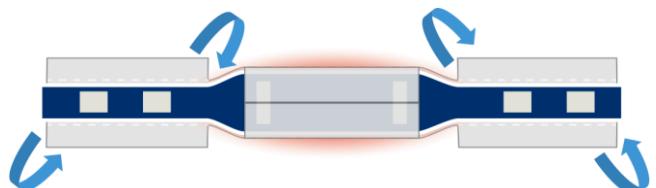Close the flaps of the lateral parts of the cover on each end of the textile. Secure with the scratches. Adjust.

**END OF SET-UP**

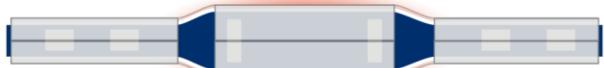The Textlight4 or Textlight6 light emitting textile with FLUXMEDICARE® Protect cover ref FMP04 or ref FMP06 is now ready for use.

*(Please refer to the user manual of the FLUXMEDICARE® medical device).*

The protective cover is for single use; it must be thrown away after each use.

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